



# Mecca Care 32mm DDA Accessible Toilet Grab Rail Set 45 /90 Degree

# Nero

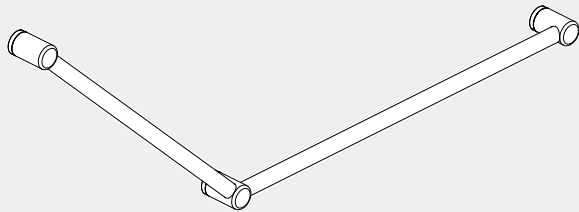


45 degree 900\*750mm - SKU: NRRCR3245

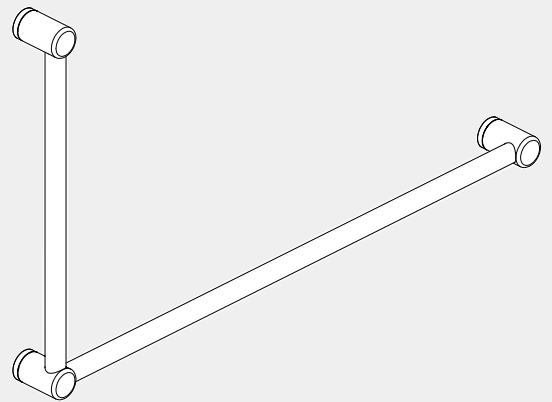

90 degree 1000\*600mm - SKU: NRRCR3290

Versions:202503

## Drawing

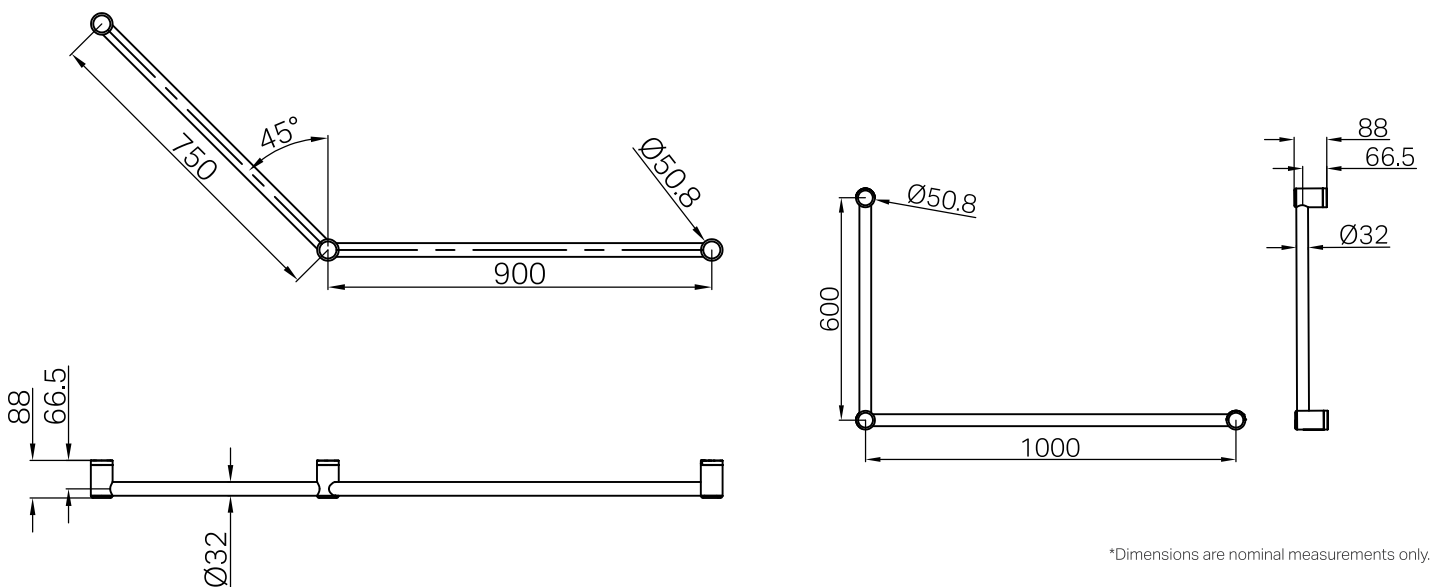







\*Dimensions are nominal measurements only.

## Warning:

These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard. It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the rail which can result in injury to user and damage to the wall.

## Finish & SKU

- |  |  |
|--|--|
|  <b>Chrome</b><br>NRRCR3245CH/NRRCR3290CH         |  <b>Gun Metal</b><br>NRRCR3245GM/NRRCR3290GM      |
|  <b>Matte Black</b><br>NRRCR3245MB/NRRCR3290MB    |  <b>Brushed Gold</b><br>NRRCR3245BG/NRRCR3290BG   |
|  <b>Brushed Bronze</b><br>NRRCR3245BZ/NRRCR3290BZ |  <b>Brushed Nickel</b><br>NRRCR3245BN/NRRCR3290BN |

# INSTALLATION INSTRUCTIONS

<p>make hole positions</p> <p>SIDE VIEW</p>	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall refer left picture. (minimum 2 supplied screws per bracket).</p> <p>Tip: Rail height is 790mm above the ground.</p>
<p>wall plugs</p>	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 3mm hole for screws into timber or 8mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Slide rail tubes into end brackets and push onto wall brackets, refer to left picture.</p> <p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p>
	<p>6. Tighten grub screws with allen key.</p> <p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p>
<p>90 Degree Set</p> <p>45 Degree Set</p> <p>7. Finished.</p> <p>SIDE VIEW</p> <p>FRONT VIEW</p>	<p>Note: The same rail can be mounted on either left or right hand side.</p>

# Nero



MECCA CARE  
DDA CARE 32MM

